GUIDOR[®] *easy-graft*

GUIDOR® *easy-graft*

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
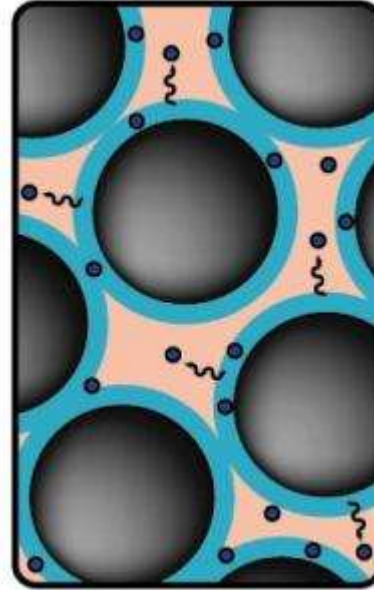
In contact with blood:
extraction of BioLinker®
material hardens

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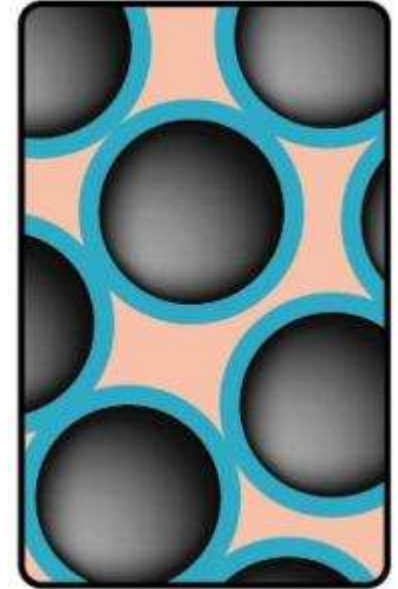GUIDOR® *easy-graft* Typical applications

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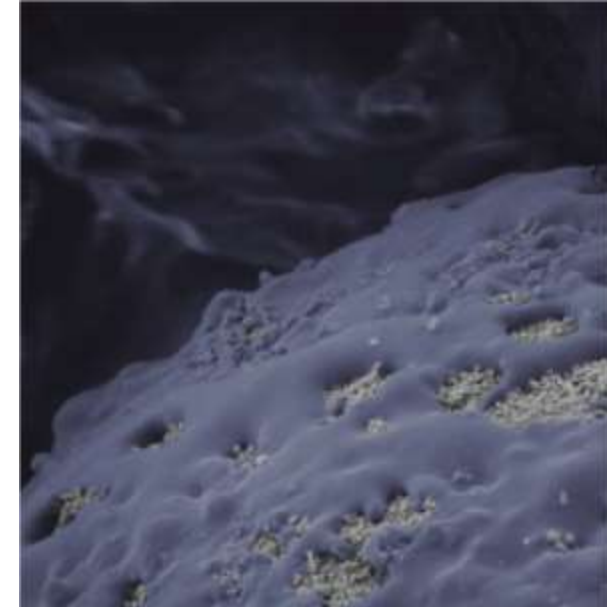
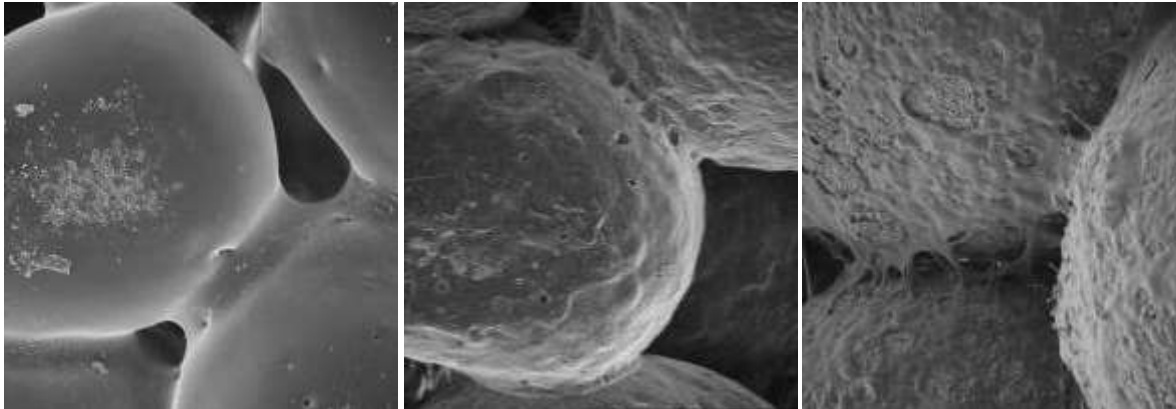
GUIDOR® *easy-graft* Resorption: Three stages

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Colored EM image. PLGA coating (blue) resorbs, exposing the granule (white).

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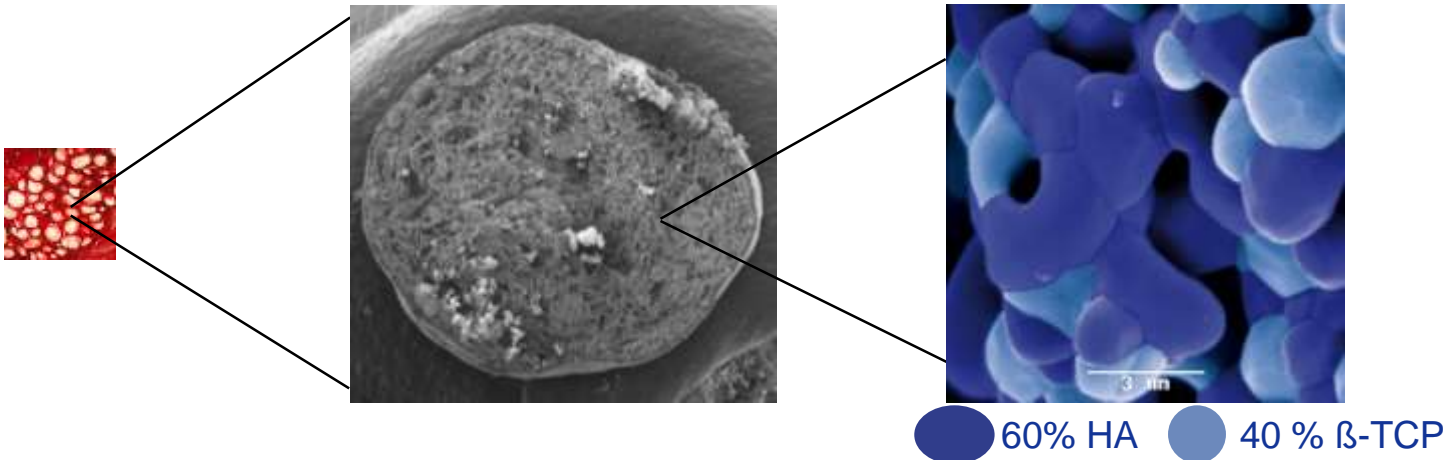
For implantation after 6 months, the authors of Post Extraction Alveolar Ridge Preservation Protocol suggest using *easy-graft* CRYSTAL.

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
The authors of Post Extraction Alveolar Ridge Preservation Protocol recommended *easy-graft CLASSIC* (β -tricalcium phosphate, resorbable) in case of early implantation (e.g “delayed immediate implantation” after 6-8 weeks).

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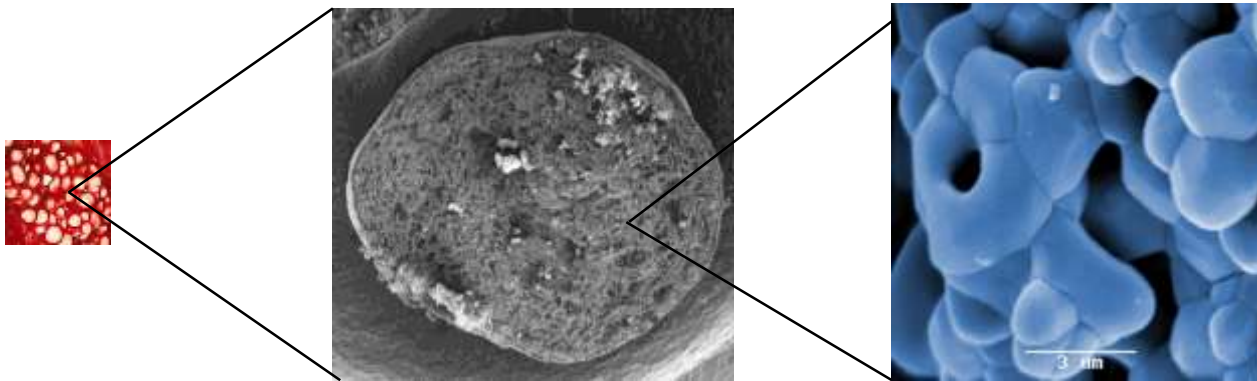● β -TCP

When?

In the treatment protocol with *easy-graft* CRYSTAL, implantation is recommended after 6 to 9 months if one application of materials was used (0,4ml). If using two applications (about 0,8ml), implantation is recommended after 9 months.

The generally recommended implantation time is intended for orientation purposes. When choosing the time of implantation, please take into account individual local and systemic factors (e.g patient age, defect size, condition of buccal plate). Implantation times can vary widely depending on the implantation protocol (e.g. in delayed immediate implantation: implantation after 6-8 weeks).

easy-graft CLASSIC is recommended in case of early implantation (e.g. „delayed immediate implantation“ after 6-8 weeks).

Visibility of granules

easy-graft CRYSTAL is partially resorbable; the granules remain integrated in the hard tissue. *easy-graft* CLASSIC is fully resorbable.

Unlike bovine materials that look similar to bone, these granules are easily visible when opening the site. The integrated granules should not be removed and can be in contact with the implant surface without problems.

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GUIDOR® *easy-graft*

Pictures of application

Application of GUIDOR® *easy-graft* directly from the syringe.

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day 4

day 10

Membrane-free application without primary wound closure. For details see „Post Extraction Alveolar Ridge Preservation Protocol“.

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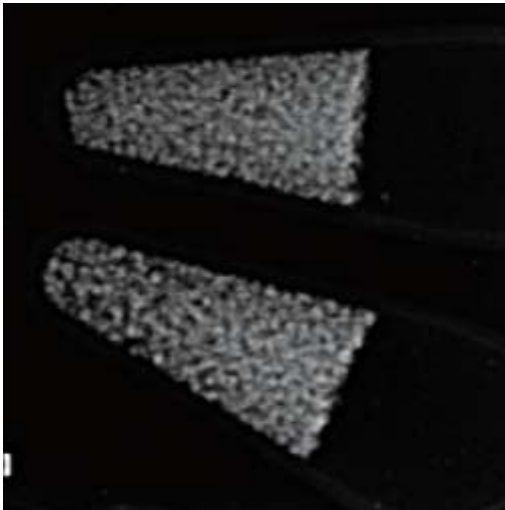
GUIDOR® *easy-graft*

Both materials, *easy-graft* CLASSIC and *easy-graft* CRYSTAL, are radiopaque.

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easy-graft CRYSTAL

MATERIAL PROPERTIES

What raw materials are used to manufacture GUIDOR® *easy-graft* products?

GUIDOR® *easy-graft* products are alloplastic materials manufactured from mineral raw materials. They do not contain any substances of animal or human origin.

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What is the difference between *easy-graft* CLASSIC and *easy-graft* CRYSTAL?

easy-graft CLASSIC contains phase pure β -tricalcium phosphate (β -TCP) and is resorbed over a period of 5–15 months. *easy-graft* CRYSTAL contains biphasic calcium phosphate (60 % hydroxyapatite, 40 % β -TCP). It is partially resorbable, the hydroxyapatite remains embedded in the new bone.

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easy-graft CLASSIC and easy-graft CRYSTAL – How do I decide which material is suitable in a specific case?

Both materials are approved for identical indications. The topic of material selection is a matter of clinician and surgical planning preference. See timing/staging of dental implant.

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easy-graft CRYSTAL: Are there two types of granules (HA and β -TCP) in easy-graft CRYSTAL?

No. Every single granule consists of a compound of 60 % hydroxyapatite and 40 % β -TCP.

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Do GUIDOR® easy-graft products have bacteriostatic properties?

A test on bacterial nutrient medium confirmed reduced bacterial growth around GUIDOR® easy-graft samples. This effect was attributed to the BioLinker®. In the body, the BioLinker® is extracted from the GUIDOR® easy-graft materials after just a few hours, therefore, the clinical relevance of the bacteriostatic properties observed in vitro is not confirmed.

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APPLICATION

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Mixing with BioLinker®: How long should GUIDOR® *easy-graft* granules be in contact with the BioLinker® in the syringe?

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Can GUIDOR® *easy-graft* products be used without mixing them with BioLinker®?

Yes, in this case the material properties are comparable with those of traditional particulate bone graft substitutes such as GUIDOR® *calc-i-oss*.

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Can GUIDOR® *easy-graft* products be dispensed?

GUIDOR® *easy-graft* products can be dispensed after mixing on a dry, sterile surface with dry, sterile tools. Under these conditions the material remains mouldable, because it does not harden until it comes into contact with aqueous liquids such as blood.

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How can a flexible, porous layer be formed with GUIDOR® easy-graft products?

Thin, flexible layers (e.g. as resorption protection for autogenous bone) can be moulded between two sterile, smooth and dry surfaces (glass, metal, no plastics) or with dry, sterile gloves with the addition of a few drops of sterile water or sterile saline solution. After contact with the material, the gloves should be changed.

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Should defects be overfilled?

No, the material swells after application. Therefore, overfilling is not recommended.

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Should the material in the defect be condensed?

GUIDOR® easy-graft products should be well condensed in the defect before the material hardens. The round granules withstand compression, therefore, fragments from fractured granules will not form.

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How should GUIDOR® *easy-graft* material be condensed?

Experienced GUIDOR® *easy-graft* users use various aids such as flattened stoppers or the plunger of the GUIDOR® *easy-graft* applicator syringe. Over larger areas the material can be evenly condensed by pressing down a piece of gauze (moistened with physiological saline solution) with the finger for 10–30 seconds.

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Can GUIDOR® *easy-graft* products be used in combination with dental membranes?

Yes, an ideal alloplastic option would be GUIDOR® *matrix-barrier*.

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When would the use of a membrane be recommended?

GUIDOR® *easy-graft* products are stable and do not require a membrane for containment in 3 or 4 walled defects. Flat (non-concave defects with limited walls) and defects of a critical size may require the additional support of a barrier membrane. Sites where a full thickness periosteal relieving flap is created may also benefit from a barrier membrane for exclusion of soft tissue ingress. The decision to use a membrane is part of therapy planning and is the responsibility of the practitioner.

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Can GUIDOR® *easy-graft* products be used in combination with autogenous bone?

Yes. For example, GUIDOR® *easy-graft* products can be used as resorption protection for autogenous bone grafts. However, bone chips should not be mixed with the material in the syringe.

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Can GUIDOR® *easy-graft* products be mixed with autogenous bone or bone graft substitutes in the application syringe?

No, mixing GUIDOR® *easy-graft* products with autogenous bone chips or foreign materials will cause the material to harden prematurely in the syringe, or will prevent the material from hardening in the defect. This means that GUIDOR® *easy-graft* products will lose their unique handling advantage. GUIDOR® *calc-i-oss* products are ideal to mix with autogenous bone or other bone graft substitutes.

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Can GUIDOR® *easy-graft* products be used with preparations such as BMP-2 and Enamel matrix proteins?

Mixing with protein preparations may result in premature hardening. The combination of GUIDOR® *easy-graft* with active biological factors in the defect has not been clinically tested. GUIDOR® *calc-i-oss* products are ideal to mix with BMP-2 and Enamel matrix proteins.

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Can GUIDOR® *easy-graft* be used in combination with antibiotics?

The use of antibiotics is the responsibility of the practitioner. Any interaction between systemically administered antibiotics and GUIDOR® *easy-graft* products is not known and not to be expected.

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Does GUIDOR® *easy-graft* adhere to the bone surface?

No. GUIDOR® *easy-graft* products do not adhere to tissue and do not contain adhesives. The granules adhere to one another and form a mouldable mass because of the coating of the granules with PLGA ("sticky granules").

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Can GUIDOR® *easy-graft* products be ground down after hardening?

Grinding down is not recommended. The effect of the rotary forces may cause the graft to loosen in the defect, which may endanger the bone regeneration. Excess material should be removed before hardening (e.g. with a curette).

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INDICATIONS

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Can GUIDOR® *easy-graft* products be used for closure of oroantral communications?

Yes, but it is important to ensure that material is not pressed into the sinus cavity.

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Is it necessary to cover the material with soft tissue after socket grafting?

No, the material will also heal in place without a soft-tissue cover. The material surface should be well condensed during socket preservation. The application of retention may be useful depending on the shape of the extraction socket. A temporary restoration serves to protect the graft surface from the tongue and foodstuffs. For examples of applications see the Sunstar GUIDOR® guidebook for ridge preservation.

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When can an implant be placed after using GUIDOR® *easy-graft* products to fill the extraction sockets?

GUIDOR® *easy-graft* products are osteoconductive bone graft substitutes. The time of implant placement can be selected in accordance with experience with comparable materials (e.g. β -TCP granules, bone replacement materials of bovine origin). A definite answer to this common question cannot be given, because the regeneration of bone depends on the anatomical and physiological conditions at the extraction site and the time of implant placement depends on the treatment philosophy.

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Can an implant be placed immediately after filling a defect with GUIDOR® *easy-graft*?

Yes, periimplant gaps and bone deficiencies around implants with primary stability can be filled with GUIDOR® *easy-graft* products. However, implant placement immediately after filling a defect is not recommended.

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Can GUIDOR® *easy-graft* products be used for fixing implants without primary stability?

No. Implants must be anchored in local bone with primary stability. GUIDOR® *easy-graft* products are suitable for filling bone deficits around implants anchored in pristine bone.

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To what degree can an atrophied alveolar ridge be restored with GUIDOR® *easy-graft* materials?

The alveolar ridge should only be grafted with appropriate techniques (bone spreading, distraction, suitable methods for onlay grafting). The possible degree of horizontal and vertical grafting depends primarily on the selected method. Simple placement of bone graft substitutes on cortical bone surfaces is not a suitable method for establishing sufficient bone volume for implant placement.

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Are GUIDOR® *easy-graft* materials radio opaque?

Yes, *easy-graft* CLASSIC and *easy-graft* CRYSTAL are both opaque to x-rays.

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How long do GUIDOR® *easy-graft* products remain stable in the body?

The adherence of the granules is determined by the PLGA coating. It is resorbed over a period of 3–6 weeks. During this period the strength of the material gradually decreases.

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Does the volume of GUIDOR® *easy-graft* products change during the healing process?

GUIDOR® *easy-graft* materials increase in volume in the first few days after application by water absorption, the material swells.

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After a defect has been filled with a GUIDOR® *easy-graft* product, some granules are expelled from the healing soft tissue during the first few days. What does this mean?

The post-operative exfoliation of individual GUIDOR® *easy-graft* granules through the soft tissue has been observed and is not necessarily a sign of suboptimal healing. However, exfoliation may also occur as the result of an infection, therefore the patient should be thoroughly examined.

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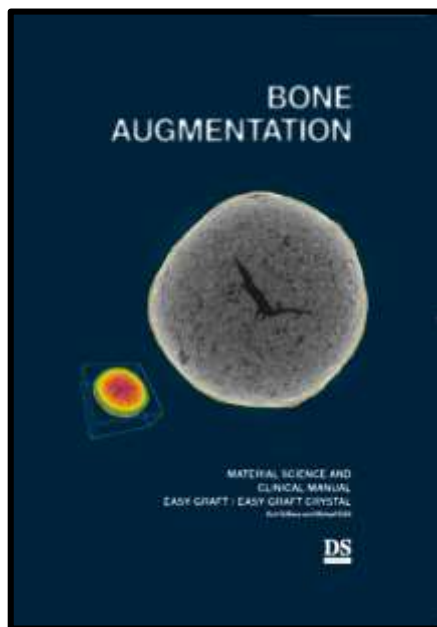
This publication is designed for dentists who are familiar with surgical protocols. General surgical treatment guidelines must be observed and are not explicitly discussed. Medical science is dynamic and constantly advancing. The presented information is accurate to the best of the authors' knowledge and reflects current knowledge at the time of publication (November 2014). The document has been written for a professional audience that is able to place the information in the proper context and assess the risks and advantages of the procedures and methods mentioned. It must be taken into account that indications differ among patients. Treatment success significantly depends on multiple biological and medical factors as well as on adequate preliminary and follow-up treatment. Any liability for material or immaterial damage arising from the use (or disuse) of this information is excluded. The *easy-graft* CLASSIC and *easy-graft* CRYSTAL instructions for use leaflets, which are authoritative regarding therapeutic information, must be observed. Each user is asked to study the instructions for use in detail.

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GUIDOR is a registered trademark of Sunstar Suisse S.A. in the U.S. and in other countries.
GUIDOR and *easy-graft* are trademarks of Sunstar Suisse S.A. in the U.S. and in other countries.