

HAMA-INSTRU

Instruments Cleaner & Disinfectant

CONCENTRATED 2%

منظف ومطهر الأدوات

Features

HAMA-INSTRU is a highly effective concentrated liquid for cleaning, disinfection, and pre-sterilization of all dental surgical instruments including burs, root canal instruments, forceps, probes, and mirrors.

Effective spectrum

Works as Bactericidal, Virucidal, and Fungicidal.

Instruction of Use

- Add 20 ml of concentrated disinfectant to 1000 ml of water.
- Soak the dental instruments for 30 - 45 minutes.
- Rinse the dental instruments using water and if there are any precipitates, remove it by brush and after that keep it on the cellulose paper for drying.

Precautions

- Ensure appropriate gloves are worn.
- Change the solution once a day or when the color changed.
- Keep out of reach of children.
- Avoid contact with eyes and if it happens, rinse immediately with water.

Composition

Aqua Water, Benzalkonium Chloride, Chlorhexidine digluconate, Betaine, Corrosion inhibitor, Aldehyde and Phenol free.

Hama Code:1111- 007

5L

المصنع كل الحماية للصناعة
Kol Al-Hamaya
Factory for Industry

P.O.Box:26316, Riyadh:11486, Saudi Arabia
Tel:+966(11)4458548, Fax:+966(11)4915906

Made in KSA صنع بالمملكة العربية السعودية

الوصف

مطهر الأدوات سائل مركز قوى ذو كفاءة عالية لاستخدامات التنظيف والتطهير وما قبل التعقيم لكل أدوات الاسنان مثل المثاقيب وأدوات الخلع وأدوات الحشو والادوات الجراحية وغيرها من الأدوات الدقيقة.

التأثير والفاعلية

يقتل معظم أنواع البكتيريا والفيروسات والفطريات.

طريقة الاستعمال

- يتم وضع ٢٠ مليلتر من المحلول المركز في ١٠٠٠ مللتر ماء.
- يتم وضع الادوات المراد تطهيرها في المحلول المخفف لمدة ٣٠-٤٥ دقيقة.
- يتم شطف الادوات بواسطة ماء نظيف وفي حالة وجود رواسب عالقة على الادوات يتم استخدام فرشاة لازالة الرواسب ثم وضعها على ورق السليلوز حتى تجف.

الاحتياطات

- تأكد من ارتداء القفاز قبل الاستخدام.
- جدد المحلول مرة واحدة يومياً أو في حالة تغير لون المحلول.
- يحفظ بعيداً عن متناول الأطفال.
- تجنب ملامسة المحلول للعين وإن تم الملامسة تشطف جيداً بالماء.



CE

HAMAYA®

